FECG Legplate Adapter for Neoventa

REF ACC300110

FECG Legplate Adapter for Philips

REF ACC300120

Table of Content

EN	Instructions for Use.....	4
FR	Conditions d'utilisation.....	7
DE	Gebrauchsanweisung.....	9
NL	Gebruiksaanwijzing.....	11
ES	Instrucciones de uso.....	14
PT	Instruções de utilização.....	16
IT	Istruzioni per l'uso.....	18
NO	Bruksanvisning.....	21
SV	Bruksanvisning.....	23
DK	Brugsanvisning.....	25
FI	Käyttöohjeet.....	28
IS	Notkunarleiðbeiningar.....	30
HU	Használati utasítás.....	32
CS	Pokyny pro použití.....	35
SK	Návod na použitie.....	37
SL	Navodila za uporabo.....	39
BG	Инструкции за употреба.....	42
HR	Upute za uporabu.....	44
RO	Instrucțiuni de utilizare.....	46
ET	Kasutusjuhend.....	49
LT	Naudojimo instrukcija.....	51
LV	Lietošanas pamācība.....	53
PL	Instrukcja użytkowania.....	56
EL	Οδηγίες χρήσης.....	58

Image 1. FECG Legplate Adapter for Neoventa


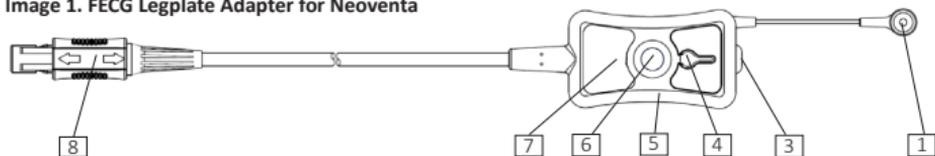
Image 2. FECG Legplate Adapter for Philips

(Backside to show difference with Neoventa version)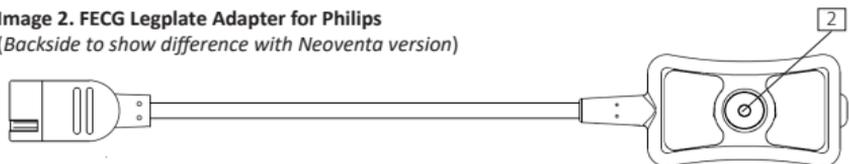

	Part names
1	Wired snap connector
2	Integrated snap connector
3	FSE port
4	Flush port/flush port plug
5	Legplate
6	Belt knob
7	Label
8	Monitor connector
9	Fetal monitor
10	Legplate belt
11	Fetal Spiral Electrode
12	Skin electrode
13	Transducer

Image 3. Neoventa

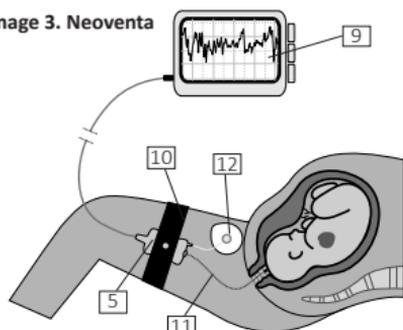
Image 4. Philips

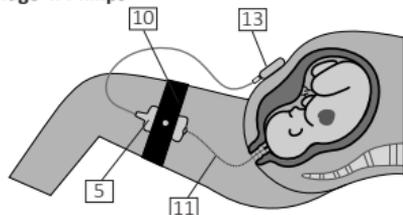
Image 5. Connecting the FSE


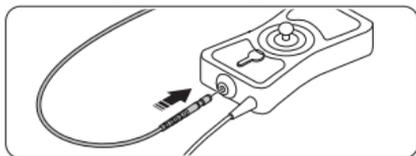
Image 6. Cleaning and Disinfecting

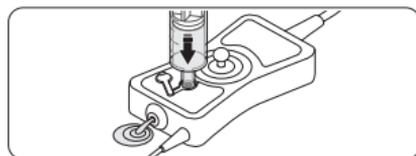Product Overview
Table 1: FECG Legplate variants and compatibility

Product	Article number	Compatible monitors
FECG Legplate Adapter for Neoventa	ACC 300 110	Neoventa monitor 
FECG Legplate Adapter for Philips	ACC 300 120	Philips monitor 
		Compatible spiral electrode
FECG Legplate Adapter for Neoventa	ACC 300 110	Goldtrace Fetal Spiral Electrode (FSE)
FECG Legplate Adapter for Philips	ACC 300 120	Goldtrace Fetal Spiral Electrode (FSE)
		Compatible Skin electrode
FECG Legplate Adapter for Neoventa	ACC 300 110	ConMed Exactrace Adult Foam wet Gel Electrodes

Device Description

The FECG Legplate is an adapter cable connecting a Fetal Spiral Electrode (FSE) and Skin electrode to a Fetal monitor for continuous electronic intrapartum monitoring.

Intended Purpose

The intended purpose of the FECG Legplate is to transfer ECG signals from a Fetal Spiral Electrode (FSE) and a Skin electrode to a Fetal monitor, for the purpose of monitoring the fetal heart rate or analysis of the fetal ECG waveform.

Indications for Use

The FECG Legplate is indicated for patients requiring continuous electronic intrapartum monitoring of the fetal heart rate or analysis of the fetal ECG waveform, during labor and delivery

Warnings

Insufficient cleaning and disinfection can lead to cross-infection and/or clogged FSE port.

Do not force the connector when connecting or disconnecting it from the monitor.

Caution

Federal (US) law restricts this device to sale by or on the order of a physician.

Intended Users

The FECG Legplate is intended to be used by medically trained personnel at delivery wards.

Intended Patient Population

The intended patient population is the fetus during delivery, and the mother.

Applying the FECG Legplate

Before you begin, please note;

- Adhere to these Instructions for Use. Failure to do so could result in poor signal quality.
- Verify that the FECG Legplate is compatible with your Fetal monitor (see Table 1).
- Connect only compatible electrodes to the FECG Legplate (see Table 1).
- Verify that the FECG Legplate has been cleaned and disinfected in accordance with instructions if it was previously used.

Connecting the FECG Legplate

1. Remove the FECG Legplate from the packaging (if first usage).
2. Press the Flush port plug into the Flush port to assure it is closed.
3. Connect the FECG Legplate to the Monitor/Transducer.
4. Prepare the skin in accordance with instructions supplied with the Skin electrode.
5. Press the Skin electrode into the Snap connector.
6. Apply the Skin electrode and the FECG Legplate with the FSE port facing towards the vagina, to the inner thigh of the mother.
7. Fixate the FECG Legplate using the Legplate belt.
8. Apply the Fetal Spiral Electrode (FSE) to the fetus, following instructions supplied with the FSE.
9. Make sure that the Connector from the FSE is clean and dry, then plug it into the FECG Legplate's FSE port (see Image 5).

Removing the FECG Legplate

1. Disconnect the FSE from the FECG Legplate.
2. Disconnect the FECG Legplate from the Monitor/Transducer.
3. Remove the FECG and disconnect the Skin electrode.
4. Dispose the FSE and the Skin electrode devices in accordance with their respective Instructions for Use.

Cleaning and Disinfection

The FECG Legplate must be cleaned and disinfected after every use.

Use only recommended cleaning solutions and disinfectants listed below (see Table 2).

Table 2: Recommended cleaning solutions and disinfectants

Cleaning Solution	Disinfectant
Water with mild soap	Ethanol 70-80 % Isopropanol 70-80%

Please refer to Instructions for Use of the Fetal Monitoring System for EMC related information.

For explanation of symbols used see Symbols glossary: <https://www.neoventa.com/symbols-glossary/>

Rx only

Not made
with natural
rubber latex

Cleaning and Disinfection of the FECG Legplate

Warning: When cleaning/flushing, always point the FSE port away from your face, to avoid being contaminated by the discharged waste.

1. Fill a syringe with a recommended cleaning solution (see table 2).
2. Lift the Flush port plug from the Flush port.
3. Hold the FECG Legplate over a sink with the FSE port facing up.
4. Insert the syringe into the Flush port and slowly inject the solution into the cavity (Image 6).
5. When the syringe is emptied, re-apply the Flush port plug.
6. Let the FECG Legplate hang over the sink with the FSE port facing up, allowing the internal cavity to soak for 5-10 minutes.
7. After 5-10 minutes, remove the Flush port plug.
8. Fill the syringe with clean water.
9. Insert the syringe into the Flush port and rapidly inject the water into the cavity, discharging the rinse water into the sink (see Image 6).
10. Repeat the clean water injection 2-3 times to assure a thorough rinse.
11. Use a soft, clean cloth with cleaning solution and wipe all surfaces clean (see table 2).
12. Change to a dry cloth and dry the FECG Legplate completely.
13. Repeat steps 11-12 using a recommended disinfectant (see table 2).
14. Let the Legplate dry with the FSE port facing down and the flush port open.

Disposal

Dispose as electronic waste according to institution's policies and procedures for safe disposal.

Reporting Responsibilities

Serious incidents related to FECG Legplate, shall be reported to the manufacturer and competent authorities without undue delay.

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