


Please note that improper operation may cause permanent damage to the Stan S41 unit.  
A software upgrade through USB is only supported from installed software 1.7.1.20180914.


## Software upgrade on Stan S41 through USB

1. Make sure that the USB stick is empty and formatted according to FAT16 or FAT32.
2. Create a “stan” folder on the USB stick.
3. Unzip the provided “upgrade.zip” file and put the “upgrade” folder (containing the upgrade files) in the “stan” folder on the USB stick.
4. Make sure the Stan S41 is connected to mains power and switched off.
5. Insert the USB stick in the USB port of the Stan S41.
6. Start up the Stan S41 and wait for the application to start. See Picture 1.
7. “Current version” are the software versions currently installed in the Stan S41, while “New version” are on the connected USB stick.
8. Identify the software to be installed, press the corresponding button and the upgrade starts.
9. When the upgrade is finished, a message will appear that the installation was successful. If installing more than one software item, please restart the Stan S41 between each upgrade procedure.
10. When finished, restart the Stan S41 and verify that the software versions were installed properly.

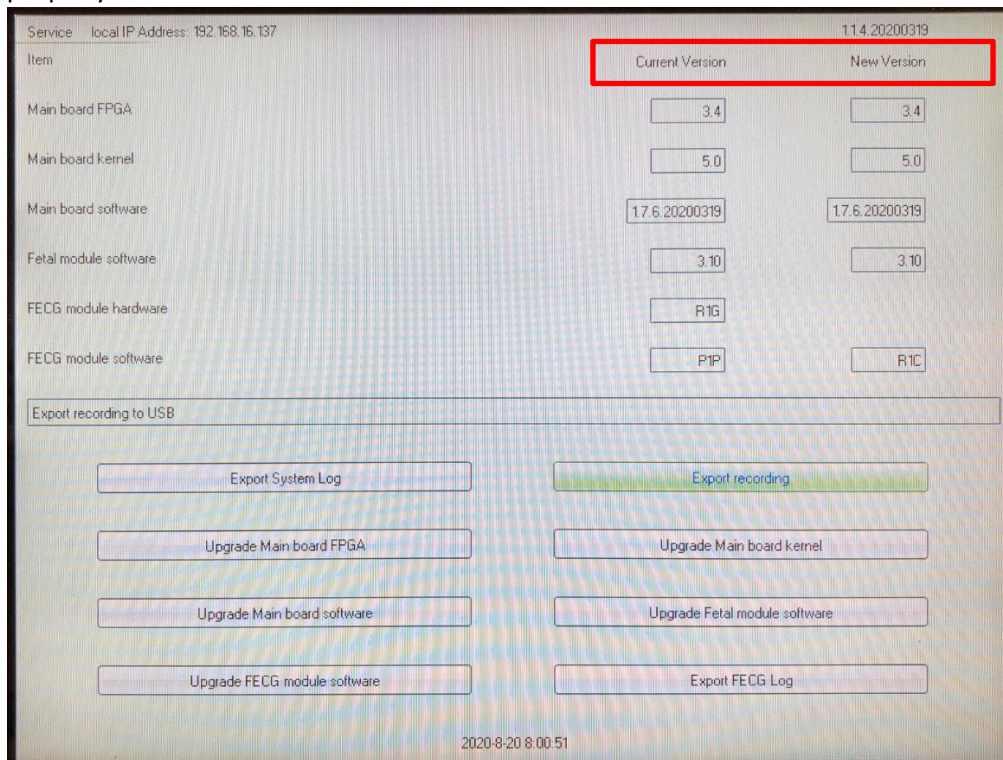

Picture 1

Fill out the Software Upgrade Form S41 and send to Neoventa at [ts@neoventa.com](mailto:ts@neoventa.com).

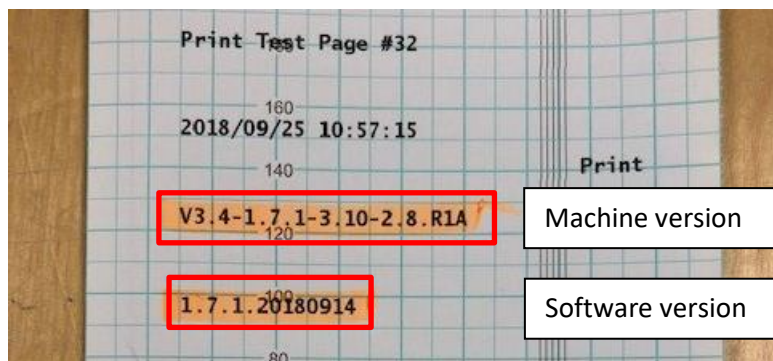

Sample of Machine version and Software version on S41 test printout

